

Operation Checklist (Ver. 02)

Machine Preparation

- 1 Place machine and scrubber into center of room.
- 2 Connect the HJ-30i and air scrubber.
- 3 Plug in, and turn on, machine and control box.
- 4 Verify deploy, dwell, scrubber, and vent times.
- 5 Check fluid level and fluid life.

Room Preparation

- 1 Identify all vents to be covered (ceiling, floor, window).
- 2 Disable or cover all smoke detectors.
- 3 Place automatic vent covers on return and supply vents.
- 4 Press "SCRUBBER ON" and raise automatic poles to seal off vents.
- 5 Once all automatic covers are raised press "CYCLE START" to let machine warm.
- 6 Seal and/or cover remaining vents.
- 7 Open closet doors, drawers, lift mattress on edge of bed, etc. to allow for fog exposure.
- 8 Identify extra equipment to be added to room for cleaning.
- 9 Remove grey fog cover from machine.
- 10 Shut and seal door to room.
- 11 Place **Altapure** sign on door and over door knob.
- 12 Start machine when status says READY.

Post Altapure Procedure

- 1 Verify cycle is complete on control box.
- 2 Remove seal from door and enter room.
- 3 Replace grey fog cover on machine.
- 4 Open all vents and remove vent covers and tape.
- 5 Drain air scrubber by connecting large drain hose under front bumper.
- 6 Drain main blower on the machine by connecting small drain hose under rear bumper.
- 7 Turn off, disconnect and remove machine.
- 8 Check floor for any condensation and wipe dry.
- 9 Reset room: close doors and drawers, remove added equipment.
- 10 Remove signs from door.

End Of Day Procedure

- 1 Store machines in storage location.
- 2 Connect charger to control box and plug in.

Altacare Filling Procedure

- Plug in machine.
- 2 Turn on machine and control box.
- 3 Verify Concentrate level shows Low or Empty.
- 4 Don protective gear: gloves, mask, eye protection.
- 5 Take off the cap of the Altacare 1 L bottle and secure the bottle adapter to the Altacare bottle by twisting.
- 6 Once secured place onto the HJ-30i.
- 7 Press "Concentrate Load" on the fluids screen.
- 8 Verify proper loading by observing bubbling inside the bottle and dropping of the fluid level in the bottle.
- 9 If needed, loosen the bottle slightly to get proper loading.
- 10 When the entire bottle has drained, press "Done" and you can remove the bottle.
- 11 If the internal tank reaches full it will automatically stop loading.
- 12 Carefully remove the adapter from the machine and then the bottle from the adapter.
- 13 Replace the cap and discard the bottle.
- 14 Only fill water to High.

Water Filling Procedure

- 1 Plug in machine.
- 2 Turn on machine and control box.
- 3 Verify Water level on the main screen of the control box.
- 4 If low, remove the black filler cap.
- 5 Add RO/DI water only until the level reads FULL *** Always use a without previous residue from another chemical.
- 6 Replace cap.

Document: JR03162015Ver02.Doc